

# GENERAL USE CONRAD® THERMAL BLANKET INSTRUCTIONS

*(Always have PATIENT SIDE facing the patient)*

# 1



## ANESTHESIA

During induction of Anesthesia have a General Use – ConRad Thermal Blanket underneath and over the patient.


# 3



## SURGERY

Cover as much of the patient as possible during surgery.

# 2

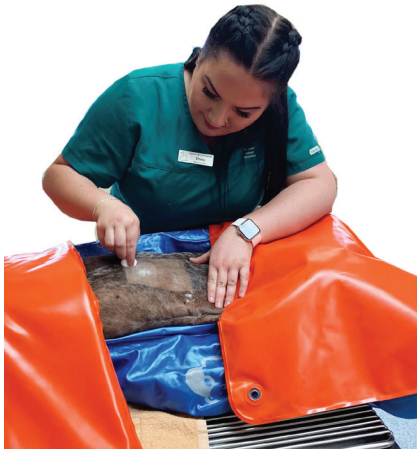

## PREP

Cover as much of the patient as possible during placement of monitors and prepping for surgery.

\*(HUG-U-VAC positioner recommended.)

# 4



## RECOVERY

Place underneath and over the patient while recovering from anesthesia, both in and out of cage.

### CONRAD Thermal Blanket For General Use:

- Surgery
- Dental
- Endoscopy
- Pre-op Prep
- Post anesthesia recovery
- Imaging – X-ray, CT

### VetORSolutions

3871 Concomly Rd S

Salem, OR 97306

Tel# 503-931-2932

[www.VetORSolutions.com](http://www.VetORSolutions.com)