

How to perform a Control Solution Test for your GlucoRx blood glucose monitor

1


Place a drop of the control solution onto the lid of the bottle. Wipe away this drop and place a second drop on top of the lid.

2

Place a test strip into the meter. The meter should be off when you insert the strip. Press the main button on the meter three times to select QC mode from GEN.

3

Dip the end of the test strip into the control solution at a 45 degree angle and allow it to suck up the solution (like a straw).

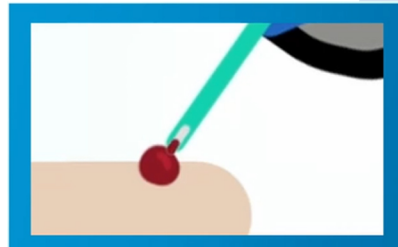

4

The meter will count down and give you a reading on the screen. Check that the result falls within the control solution test range printed on the test strip pot. If it does not, repeat and then contact GlucoRx Customer Care on 0800 007 5892.

