

Prevantics® 2% CHG Applicator: Training Poster

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.



Step 1: Open the packaging by pulling away the label from the corner.



Step 2: Firmly press the dispenser to release the liquid from the chamber onto the sponge.

DO NOT shake the applicator to mix the solution.



Step 3: For using an applicator with the sponge facing downwards, use the applicator **immediately** after pressing the dispenser.

When using the applicator with the sponge facing upwards, press the dispenser, then turn the applicator so the sponge faces downwards. Leave for the suggested times below to allow the solution to drain into the sponge before use.

3ml - 10 seconds, 10.5ml - 15 seconds, 26ml - 30 seconds



Step 4: Apply the product to the area to be disinfected using vigorous back-and-forth, up-and-down, repeated strokes for at least 30 seconds. Allow the skin to air dry.

Tips:

- + Ensure the right applicator is chosen for the correct skin coverage.
- + Do NOT dispose in sharps bin, follow local policy for waste disposal.

Applicator Size	Recommended Skin Coverage	Drainage Time
26ml	50cm x 50cm	30 seconds
10.5ml	30cm x 30cm	15 seconds
3ml	15cm x 15cm	10 seconds

Note: In the UK, Prevantics® 2% CHG Applicators are a PT1 (Product Type 1) Skin Biocide intended for skin disinfection only. It is not a licensed medicinal product and not intended for pre-surgical skin antiseptics. Use biocides safely. Always read the label and product information before use.